



Acquisition Guide **MovumStudio**

The result of the identification of dental implants by our Artificial Intelligence (AI) algorithms depends on the quality of the images provided. Therefore, you must follow these guidelines to achieve maximum accuracy in implant identification.

- 1** Use periapical xray instead of panoramic xray.

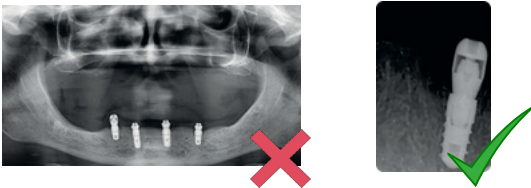


- 2** Macrogeometry should be visible over the entire length of the implant, including the platform and apex.

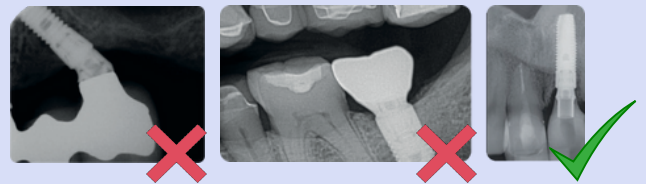


- 3** Prioritize taking xray with parallelizing systems or similar devices.




- 4** The xray must be properly exposed.

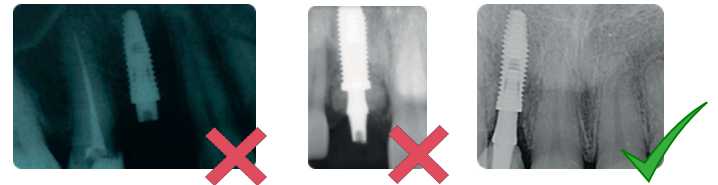

- 5** Ensure that the xray is not damaged, avoiding any type of artifact in the image that coincides with the implants to be identified.

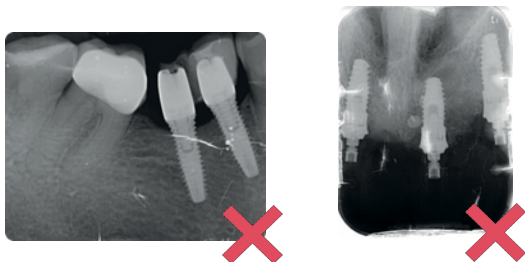

- 6** Prioritize the upload of xray exported directly from the computer, through your management software, or by a screenshot. Avoid taking photographs with a smartphone, as details could be lost.

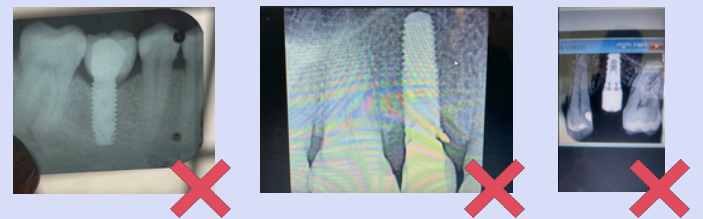

- 7** Our system can only identify up to 5 implants per xray. If necessary, crop the image before uploading.

